



MYCO
MEDICAL

TheSharpChoice®

NMSDC Certified Diversity Supplier

DATASHEET

QLICKSMART® BladeFLASK Scalpel Blade Removal System

The QlickSMART® BladeFLASK is the world's first single-handed scalpel blade remover.



- Holds 100 blades and is puncture proof
- OSHA compliant with single-handed activation when used with reusable QlickSMART® BladeFLASK Mounting Bracket (*sold separately*)
- Mount on a wall or desk with the reusable QlickSMART® BladeFLASK Mounting Bracket (*sold separately* — adhesive backing and screw mounts included)
- Audible click ensures blade removal
- Built-in counter with automatic shutoff when full
- Compatible with most brands of general surgery blades and flat handles



NO FINGERS



NO FORCEPS



NO RESHEATHING



NO INJURY

MYCO CODE	DESCRIPTION	UNIT OF MEASURE
QFRGEN-01	BladeFLASK — Non-Sterile	12 ea/case
QBRGEN	BladeFLASK Mounting Bracket — Non-Sterile (Reusable, includes adhesive backing and screw mounts)	24 ea/case

How to Use:

PARTS OF YOUR BLADEFLASK:

INDICATOR: SHOWS WHICH WAY SHARP EDGE MUST FACE.

THIS PART OF THE SCALPEL HANDLE IS CALLED THE TANG.

COUNTER

LANDING PAD

CLICKSMART
Simply safer

MOUNTING BRACKET

MOUNT TO WALL OR TROLLEY:

- If you haven't mounted your BladeFLASK before, prepare mounting bracket.
 - BRACKET HAS DOUBLE SIDED TAPE ON BOTTOM;
 - OR USE SCREWS.

* See fitting template and instructions inside the bracket packaging.
- BladeFLASK can be mounted on a wall or on a trolley or bench.

NOTE: BladeFLASK is mounted on Mounting Bracket during use.

REMOVE BLADE SINGLE HANDED ACTION:

- Place blade on landing pad with the sharp edge facing to the right (or 'tang' up).
- Push scalpel handle into BladeFLASK until you hear a click.
- Remove handle.

NOTE: BladeFLASK is disposed of as medical waste after 100 blades have been inserted.

As a leading NMSDC Diversity Certified Supplier, MYCO Medical is committed to “Bringing Reliability, Innovation and Value” to the Healthcare industry with an emphasis on “physician preference” surgical devices and disposables.

To order, learn more or receive complimentary evaluation samples:

Please call MYCO Medical at (800) 454-6926 or (919) 460-2535 or email us at info@mycomedical.com



2015 Production Drive
Apex, NC 27539 USA

info@mycomedical.com
www.mycomedical.com

See the complete line of MYCO innovative disposable and reusable medical devices at
www.mycomedical.com

Some medical devices are not licensed for sale in Canada.

© 2017 MYCO Medical, Inc. All rights reserved.
LIT-00217 Rev 2 Eff 10.24.2017